Lifeguard Accessory

Monitor Shelf / Footboard - Model 157018 Installation and User Instructions

ARJOHUNTLEIGH
GETINGE GROUP

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EN

JA

Foreword

Thank you for purchasing ArjoHuntleigh equipment.

Please contact us if you have any questions about the operation or maintenance of your ArjoHuntleigh equipment.

NOTE

Means: This is important information for the correct use of this system or equipment.

EN

Definitions in this IFU

WARNING

Means: Safety warning. Failure to understand and obey this warning may result in injury to you or to others.

Means: The name and address of the manufacturer.

CAUTION

Means: Failure to follow these instructions may cause damage to all or parts of the system or equipment.

Design Policy and Copyright
© and ™ are trademarks belonging to the ArjoHuntleigh group of companies. © ArjoHuntleigh 2017.
As our policy is one of continuous improvement, we reserve the right to modify designs without prior notice. The content of this publication may not be copied either whole or in part without the consent of ArjoHuntleigh.

Introduction

The monitor shelf / footboard is designed for the Lifeguard® range of trolleys. It has a 'fold-away' feature allowing it to be stored with the trolley when not in use. (See Fig. 1) (See Fig. 2)

Fig. 1

Operating position

Fig. 2

Stored position

WARNING

Do not exceed the maximum safe load of 12 kg.

Do not install the shelf at the head of the trolley.

Installation

Remove any existing footboard that may be fitted to the trolley.

Position the monitor shelf so that it will hang inboard, over the foot of the bed, when in the operating position.

EN

Insert the monitor shelf / footboard uprights fully into the two square sockets at the foot-end of the trolley.

Locate both straps around any monitoring equipment placed on the shelf and adjust to secure.

Storage

Remove monitoring equipment and fold the shelf up and over so that it hangs down in front of the footboard. Refer to illustration showing stored position.

Removal

Remove monitoring equipment and lift the monitor shelf / footboard out of its sockets.

Decontamination

WARNING

Gloves and protective clothing should always be worn when carrying out cleaning procedures.

Cleaning

Clean all surfaces with a disposable cloth soaked in neutral detergent and hand hot water.

Take extra care with areas that may harbour dirt or dust.

Rinse with clean water and dry with paper towels.

Disinfecting

After cleaning as described above, wipe all surfaces with sodium dichloroisocyanurate (NaDCC) at a concentration of 1000 ppm (0.1%) available chlorine.

In the case of pooling body fluids, e.g. blood, increase the concentration of NaDCC to 10,000 ppm (1%) available chlorine.

CAUTION

Do not use abrasive compounds or pads.

Do not use phenolic cleaning solutions.

Labels

On the label you will find the following information:

Symbol explanation	
	Safe Working Load (SWL)
	Reference number
	Serial number
	CE marking in accordance with The Medical Devices Directive 93/42/EEC.
	Manufacturer name and address
	Date of manufacture

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HONG KONG

ArjoHuntleigh (Hong Kong) Limited
Room 411-414, 4/F, Manhattan Centre,
8 Kwai Cheong Road, Kwai Chung, N.T.,
HONG KONG
Tel: +852 2960 7668
Fax: +852 2960 1711

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ArjoHuntleigh is a world-leading provider of integrated products and solutions that improve the lives of patients and residents with reduced mobility. We help healthcare facilities deliver wellness and effective everyday care, early mobilisation, safe patient handling, venous thromboembolism prevention, pressure injury prevention, hygiene routines, bariatric care and diagnostics. With extensive knowledge and experience, we strive to improve efficiency and ensure a safer and dignified environment for caregivers and their patients.

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Arjo Inc.
ArjoHuntleigh AB